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Disinfection of EasyCap Caps and Electrodes

Any disinfection is more stressful for materials than simple cleaning. However, caps and electrodes will not be ruined by a single disinfection. Rather, disinfection will accelerate premature aging. Under time pressure, EasyCap may also be disinfected with alcohol – but alcohol will enhance the plastics becoming brittle, more than compared to our recommended instrument disinfection agent:

for EU: Perfektan TB

outside EU:Metricide (plain)

Please note: dry air by itself is a very good disinfectant. Most viruses and bacteria will not survive in dry environment at room temperature for more than a few minutes. If one has to decide on disinfection, please keep in mind that even if working with infectious patients, or patients from at-risk-groups, it is less the next participant who is in danger of getting infected. Rather, the persons mounting the cap and minimizing impedances are the ones at risk, as they are more prone to get in touch with body liquids. Protection here is not a question of disinfection but of other actions: do not work with open injuries at fingers, nailbeds, or at mucous membranes, wear surgical gloves and respiratory protection mask, and get information on the risks of infections.

When disinfecting caps or electrodes,

- Please only apply the minimum concentration and the minimum of soaking time required. Make sure that the cap/electrodes are fully covered with the disinfectant. Please do not leave the materials in the disinfecting solution longer than required, for extended period of time.

- After disinfection, please rinse the materials with water.

Finally, a brief remark on the legal situation: the responsibility for hygienic control is always with the person in charge for the laboratory. In case of any ambiguities or concerns on the level of disinfection required, please contact your local department responsible for hygienic matters.